Manual breast pump Tire-lait manuel Manueller Milchpumpensatz Handmatige borstpomp Tiralatte manuale Sacaleches manual Tira-leite manual

Instructions for use Conseils d'utilisation Gebrauchsanleitung Gebruiksaanwijzing Consigli per l'utilizzo Consejos de uso Conselho de utilização

DOREL FRANCE S.A.S. - Z.I. - 9 bd du Poitou, BP 905, 49309 Cholet Cedex, FRANCE
 DOREL UK 4 Imperial Place, Maxwell Road, Borehamwood, Hertfordshire, WD6 1JN, UNITED KINGDOM
 DOREL GERMANY GMBH Untergasse 9, 50667 Köln, DEUTSCHLAND
 DOREL ITALIA S.P.A. a Socio Unico - Via Verdi, 14, 24060 Telgate (Bergamo), ITALIA
 DOREL BENELUX - P.O. Box 6071, 5700 ET Helmond, NEDERLAND
 DOREL HISPANIA S.A.U. - Edificio Barcelona Moda Centre, Ronda Maiols, 1, Planta 3ª, Local 340, 08192 Sant Quirze del Vallès, ESPAÑA
 DOREL PORTUGAL - Rua Pedro Dias, 25, 4480-614 Rio Mau (VDC), PORTUGAL

www.bebeconfort.com

160 mm

180 mm

REF. 31021000
01402027

EN - MANUAL BREAST PUMP

Congratulations on your purchase of this Bébé Confort breast pump. Read this instruction leaflet carefully in order to use the product correctly. We advise you to keep it, to be able to refer to it later if necessary.

- A • PARTS**
- | | |
|---------------------------------|--|
| 1 - Intensity adjustment button | 6 - Silicone end piece |
| 2 - Handle | 7 - Non-return valve |
| 3 - Rod | 8 - Adapter for Bébé Confort Perfect Sense bottles |
| 4 - Diaphragm | 9 - 150 ml feeding bottle |
| 5 - Pump | |

Breastfeeding is the easiest, most natural way to feed your baby and contribute to his/her development. Breast milk is perfectly adapted to Baby's needs and contains antibodies which protect him/her against illness and infections.

B • CLEANING AND/OR STERILISING

Before using for the first time, immerse all the breast pump parts in boiling water for 10 minutes.

Before use always clean and/or sterilise all the parts of the breast pump which you have dismantled (see Paragraph C). You can clean the different parts in hot soapy water or sterilise them in boiling water for 5 to 10 minutes, or in a steam steriliser.

Caution:

- Do not sterilise the various parts in the microwave as there is a risk this might damage them.
- Allow the parts to cool before using them again.
- Once cleaning and/or sterilisation has been carried out, observe the following recommendations for use of the breast pump (see Paragraph D).

C • ASSEMBLING/DISASSEMBLING THE BREAST PUMP

To assemble/dismantle the breast pump refer to the diagrams opposite.

D • USING THE BREAST PUMP

- 1 - Wash your hands carefully before using.
 - 2 - Move the intensity adjustment button as far as possible towards the minus symbol to start at the lowest level (see the diagrams opposite).
 - 3 - This breast pump offers two suction modes: Stimulation mode (mode A) imitates the movement of your baby when he starts to suck to encourage the milk to flow. Expression mode (mode B) for deeper, longer suction when the milk has started to flow. To select the suction mode (A or B), change the position of the handle by pushing the button away from the "U" shape of the handle (see the diagrams opposite).
 - 3 - Position the handle of the breast pump on the Stimulation mode (see mode A on the diagram opposite).
 - 4 - Then place the silicone end piece on your breast around the nipple, and pump for a few seconds to encourage the milk to flow. Make sure the silicone end piece adheres to your breast to ensure there is no air suction on the sides.
 - 5 - Make sure the machine is in the vertical position. A slanting position may cause the milk to flow into the pump.
 - 6 - Once your milk has started to flow, position the handle of the breast pump on the Expression mode (see mode B on the diagram opposite).
 - 7 - Press regularly on the pump handle and release it slowly.
 - 8 - Move the suction adjustment button to the level you find effective and comfortable (see diagram opposite).
- Do not forget to clean and/or sterilise your breast pump before using (see Paragraph B). You can express milk several times a day from each breast if necessary. Each mother's morphology is different and the time for expressing milk can vary between 10 and 20 minutes per breast.
- An inexperienced mother may feel she has not expressed enough milk at the start, but with time she will find the best way of expressing her milk.

E • STORING EXPRESSED BREAST MILK

- Pour the milk into a clean air-tight container which can be closed and dated, or use the

feeding bottle as a storage container using a Bébé Confort cap.
 - For milk storage times, refer to the table below.
 - When you are ready to use the milk, remove it from the refrigerator or freezer. Defrost the milk in the refrigerator for a few hours, then at room temperature. You can warm it up if you wish. Roll the bottle between your hands to ensure the temperature is uniform.

	Storage time at room temperature (20°C)	Storage time in a refrigerator	Storage time in a freezer compartment	Storage time in a freezer
Freshly expressed milk	6-10 hours	3-4 days at 4°C	2 weeks at -12°C	Around 6 months at -18°C
Defrosted milk	1 hour	10 hours at +4°C	Do not refreeze once defrosted	Do not refreeze once defrosted

Only store milk expressed with a sterilised breast pump for your baby. NEVER refreeze breast milk once it has been thawed. Never add freshly expressed milk to frozen milk. Always throw away any milk your baby leaves at the end of a feed. Do not keep the milk in the door of the refrigerator (the temperature there is higher than in the rest of the refrigerator).

F • TIPS AND TRICKS

PROBLEM	SOLUTION
Lack of suction	Check that all parts of the breast pump are assembled correctly and that the valve is pushed in properly. Check that no air is trapped between the silicone tip and the pump.
The milk does not come out	Check that the breast pump is able to suck and that all parts have been correctly assembled. Don't worry - try to relax. The amount of milk can vary from day to day; you may have to wait a while on your first few tries. You may find this advice helpful: <ul style="list-style-type: none"> • Relax and find a place where you will not be disturbed. • Keep your baby near you, or look at a photo. This will help to release the milk. • You could apply a soothing Bébé Confort compress or put a warm, damp flannel on your breasts for a few minutes. This will help to release the milk and soothe your breasts. • You could also swap to the other breast.
Breast pain	Decrease the suction level, halve the pressure on your breast, and release slowly.
Parts show signs of wear	Avoid putting the parts into contact with detergents that may damage the plastic.

INSTRUCTION STORAGE BAG FOR BREAST MILK USE

- Wash your hands carefully before starting to transfer the milk
- Remove the pre-cut part at the top of the bag along the dotted line and throw it away.
- Open the double zip at the top of the bag.
- Fill the bag with breast milk, but do not go over the 150 ml mark. The scale of measurements given on the bag is an indication only. For more precise information about volume, refer to the quantity shown on the feeding bottle before pouring the milk into the bag.
- Smooth down the bag above the milk to get rid of any air and close the bag using the double zip.
- You can write the date and quantity of the milk expressed on the label (at the top of the bag).
- The bag can only be used once and should be thrown away once empty.
- Frozen milk can be kept at a temperature of 4°C for up to 24 hours without opening the bag.
- To heat the milk, pour it into a container which is suitable for heating. WARNING! The bag cannot be put in a microwave. WARNING! Always check the temperature of the food before feeding your child.

BABY BOTTLE USE

- Always wash the teat before and after use.
- Wash the teat regularly in soapy water, then rinse thoroughly.
- Store teats in a cool and dry place away from light.
- Remove the teat before warming the bottle.
- Never pierce the teat.

For your child's safety and health

- WARNING!**
- Continuous and prolonged sucking of fluids will cause tooth decay.
 - Always check food temperature before feeding.
 - Throw away at the first signs of damage or weakness.
 - Keep components not in use out of the reach of children.
 - Never attach to coats, ribbons, laces or loose parts of clothing. The child can be strangled.
 - Always use this product with adult supervision.
 - Never use feeding teats as a soothe.
 - Before first use, disassemble and clean the product and then place the components in boiling water for 5 min. This is to ensure hygiene.
 - Before each subsequent use clean carefully to ensure hygiene.
 - Localised over heating can occur. Always mix the heated food and check the temperature before feeding.
 - Inspect the feeding teat before each use and pull the feeding teat in all directions. Throw away at the first signs of damage or weakness.
 - Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant ('sterilising solution') for longer than recommended, as this may weaken the teat.
 - Do not use if there are signs of damage or weakness to either the primary packaging or the product.
 - Glass bottles may break.
 - Conforms to EN 14350.

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