Discreet

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Wearable electric breastpump

Tragbare elektrische Milchpumpe

Handenvrije elektrische borstpomp

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EN - Discreet wearable electric breastpump

Congratulations on your purchase of this Bebeconfort Discreet wearable breastpump. Read this instruction leaflet carefully to use the product correctly. We advise you to keep it, to be able to refer to it later if necessary.

Breastfeeding is the easiest, most natural way to feed your baby and contribute to their development. Breast milk is perfectly adapted to baby's needs and contains antibodies which protect them against illness and infections.

The Discreet wearable electric breastpump from Bebeconfort will provide you with the efficiency and comfort of use you need while having the freedom to express anytime, anywhere.

Please read the instructions in full before using the breastpump.

Keep these instructions for future reference.

Warnings identify all instructions that are important for safety. Failure to observe these instructions can lead to personal injury or damage to the product.

The following symbols and signal words show the significance of the warnings:

WARNING Can lead to serious injury.

CAUTION Can lead to minor injury. Can lead to material damage (not related to personal injury).

WARNING

During charge, the socket-outlet shall be installed near the breastpump and shall be easily accessible.

To avoid fire, electric shock, or serious burns:

- Only use a specified power adapter (DC 5C ≥ 1A). The power adaptor is not provided. Only use the USB cable that comes with the device. An inadequate power source or cable may result in a fire hazard, electric shock or malfunction of the device.
- Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. If damage is found, immediately discontinue use of power adapter and contact Bebeconfort.
- Do not use an electrical product that has been exposed to water or other liquids including:
 - ◊ Do not use while bathing or showering.
 - ◊ Never place or drop into water or other liquids.
 - ◊ Do not run water over the pump unit.
 - ◊ If an electrical product has been exposed to water or other liquids, do not touch, unplug the device from electrical outlet, turn off and contact Bebeconfort.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.
- Keep the device dry and protect it from direct sunlight, and never leave in the car as high temperature will damage the battery and could cause fire
- To avoid health risks and reduce the risk of injury:
 - ◊ Do not use the breastpump while operating a moving vehicle.

CAUTION

To avoid health risks and reduce the risk of injury:

- The breastpump is for personal use only. For hygiene reasons it must never be used by another person.
- Use the device only for its intended purpose as described in these instruction for use.
- This product cannot be serviced or repaired. Do not repair yourself. Do not modify the device.
- Check the various parts of the breastpump for signs of wear or damage.
- Never use a damaged device. Replace defective or worn parts.
- Only use Bebeconfort recommended parts with the device.
- Do not use the device while sleeping or overly drowsy.
- Keep the breastpump, the cable and its parts away from children aged 3 years and younger.
- Pumping can induce labour. Do not pump until after giving birth. If you become pregnant while breastfeeding or breastpumping, consult with a licensed healthcare professional before continuing.
- Do not place or store the product where it can fall or be pulled into a tub or sink.
- The device and its accessories are not heat-resistant. Keep away from heated

- surfaces or open flames.
- Do not use or store near flammable materials.

A - OVERVIEW

- 1 - Silicone cup
- 2 - Silicone diaphragm
- 3 - Linker
- 4 - Pump motor
- 5 - Type C-USB charging cable (1 meter)

- 6 - Valve (+ spare replacement one)
- 7 - 210ml milk collector
- 8 - Bra strap
- 9 - 4 sizes silicone nipple inserts (16-18-20-22mm)
- 10 - Transport bag
- 11 - Protection lid

B - CONTROL PANEL

- a - ON/OFF
- b - Change suction mode
- c - Increase suction level
- d - Lower suction level
- e - Modes

⌘ Stimulation mode

✎ Massage mode

⏏ Expression

f - Battery status

Fully charged

When the first bar start flashing, it can be still used for 2 minutes warming before requesting to be charged.

Flashing - when the pump is plugged in, the bars of the battery indicator will flash one by one until fully charged.

C - OPERATING INSTRUCTIONS

CHARGING THE BATTERY

- It must be charged before using it the first time.
- To charge the battery, connect the provided type-C end of the charging cable (part 5) into the charging port of the breastpump.
- Insert the charging cable to a power source or into a power adaptor. The power adaptor is not provided.
- Insert the power adaptor plug into an AC socket.
- Unplug the power adaptor from the socket.
- Disconnect the charging cable from the power adaptor or power source and from the breastpump after charging is completed.

WARNING

- Never use the breastpump while charging.
- If the charging cable is damaged, broken or cracked, stop charging the breastpump and contact Bebeconfort customer care to get a replacement one.
- Stop using the breastpump immediately if you notice any smoke or burning from the pump unit or power adapter.
- If the breastpump is stored for a long time, it is required to charge every six months, and the state of charge should be no less than 2 bars.

D - CLEANING AND STERILISATION

BEFORE FIRST AND EVERY USE

- For the first time, before and after each use, you must disassemble, wash and sterilise all the breastpump parts that come into contact with the breast and breast milk (parts 1, 2,3,6,7 in overview A).
- Wash your hands thoroughly before starting to disassemble or assemble your breastpump.

WARNING

- Never put the pump motor or the charging cable in water or dishwasher as this causes permanent damage.
- Never wash or sterilise the pump motor. The pump motor doesn't have direct contact with milk.

CAUTION

- Do not use antibacterial or abrasive cleaning agents to clean the breastpump parts as this may cause damage.
- Take care when cleaning the valve and silicone diaphragm. If they get damaged, your breastpump will not function properly.
- Do not use a microwave or UV steriliser to sterilise the breastpump parts.
- Do not use solvents or alcohol to wash the breastpump parts nor the motor.
- Be careful when you have sterilised the breastpump in boiling water or in steriliser as parts can be hot. To prevent burns, only start assembling the breastpump when parts have cooled down.
- If any parts becomes mouldy, clean thoroughly, if possible, otherwise discontinue use and replace them.

DISSASSEMBLING

1. Remove protection lid
2. Separate motor for milk collector
3. Detach silicone diaphragm from motor
4. Detach silicone cup from milk container
5. Remove linker (part 3) from silicone cup and remove valve

CLEANING

1. Soak the breastpump parts in warm soapy water (40-60°C) for a 3-5 minutes and use a clean gentle brush to wash off any milk residue.
2. To clean the valve, rub gently between your fingers in warm soapy water.
3. Rinse thoroughly under tap running water

CAUTION

Do not insert objects or do not use a brush to clean the valve as this may cause damage. If the valve is damaged, your breastpump will not work properly. Use the provided spare valve in case of a damaged or lost valve.

STERILISATION

Use boiling water in a pot (5-10min), an electric steriliser or cold water sterilising tablets to sterilise ONLY parts that come into contact with the breast milk (temperature not exceeding 110°C as this will shorten the life of parts)

CLEANING THE MOTOR

Wipe the motor clean with a soft damp cloth

E - ASSEMBLING

1. Press the silicone diaphragm onto the round hole of the milk collector, and press it with your hands to make the edges fasten lightly to ensure a perfect seal.
2. Put the silicone valve at the end of the linker.
3. Put the assembled linker into the milk collector, ensuring the lines on the linker fit in the grooves of the milk collector.
4. Align the pouring spout in the milk collector with the groove on the silicone shield and set in position.
5. Install the silicone cup on the milk collector by pressing the rim down with your thumbs to ensure a perfect seal.
6. Press round part of the milk collector with the silicone diaphragm onto the pump motor and ensure it fits perfectly to avoid a fall during its use. The Bebeconfort logo should face outwards.

F - PREPARING FOR EXPRESSING

Choosing the right size insert (parts 9)

- The Discreet wearable electric breastpump comes with a silicone cup with a 24mm opening plus four nipple inserts of 16, 18, 20 and 22mm to fit most women's breasts.
- To find the right size insert, first stimulate your nipple to get a full size
- Place the nipple into the smallest hole you can comfortably fit into without forcing it, either directly in the silicone cup or with an insert.
- During pumping, if your nipple moves smoothly without rubbing without touching the inner wall of the insert/cup, and almost no areola is sucked inside, then this is the right size insert.

Fitting pump in bra (Picture 1)

- To make expressing easier choose a breastfeeding bra made of stretchy fabric, ideally with no underwire or padding.
- When putting the pump inside your bra, the silicone cup should be firmly pressed against your skin without a gap, creating a gentle seal. Your nipple should be centered for an efficient and pain-free pumping session
- Keep the breastpump upright and straight to avoid loss of suction and spilling milk
- If the pump moves during use, tighten your bra strap.
- Use the provided bra strap to adjust the length of your bra and create extra space to fit your breastpump.

Sitting position for expressing (Picture 2)

- Relax your body, sit in a comfortable chair and lean slightly forward (use a cushion to support your back). Make sure to keep the breastpump and milk collector upright to avoid any spilling.

G - EXPRESSION / PUMPING

-  **Stimulation mode** (1-9 levels): slightly longer sucking pattern mimic baby's suction to stimulate milk flow
-  **Massage mode** (1-9 levels): regular quick sucking pattern to get breast ready for expression
-  **Expression** (1-15 levels): Deeper sucking pattern to express milk

STARTING PUMPING

- Lightly press the  button to start the breastpump
- Lightly press the « M » key to switch to the preferred mode.
- You can select the suction power that makes you comfortable by touching the « + » and « - » keys.
- When you have finished pumping, press the button  to turn off the breastpump, then carefully remove the breastpump from the breast. You can use your finger to break the vacuum seal between the breastpump and the breast.
- Memory function: the pump will remember and start the previous suction power level and mode when you next use it.
- A pumping session lasts approximately 10-15min per breast, but you should only pump as long as it is comfortable and efficient for you.
- We recommend to stop pumping once the volume of expressed milk approaches 180ml to avoid overflow.

CAUTION

- Before expressing breast milk, you should wash your hands with soap and water or an alcohol-based hand sanitizer. The pump parts, bottles and pumping area must be clean. Breasts and nipples do not need to be washed before pumping.
- Before removing the pump from your breast, always switch off the breastpump to release the vacuum seal.
- Do not continue pumping for more than 5 minutes at a time if you do not succeed in expressing any milk. Try to express at another time during the day.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast shield with your finger and remove the breast shield from your breast.
- While some discomfort may be felt when first using a breastpump, it should not cause pain. If it becomes very uncomfortable or painful, stop using the pump and consult your lactation or healthcare professional.

H - Pouring and storing breast milk

Pouring breast milk

- Please keep the breastpump upright, and remove the pump motor from the milk container carefully
- Pour the milk into a bottle (or container) using the pouring spout of the milk collector.

Storing breast milk guidelines (see below table)

- Store the breast milk in a clean and sterilised storage bag or sealed bottle and put them in the fridge or freezer immediately.
- Do not fill the bottles or bags more than 3/4 full to allow space for possible expansion.
- Label the bottles or bags with the date and volume of expression.

Unfreeze breast milk

- To preserve breast milk components, put the frozen breast milk in the fridge to unfreeze overnight.
- In case of any emergency, you can defrost the milk in a bottle warmer to heat the milk quickly or hold the bottle or bag under warm water (max, 37°C)

CAUTION

- Only store milk expressed with a sterilised breastpump for your baby.
- Always throw away any milk your baby leaves at the end of a feed.
- Store breast milk in the coldest spot of the refrigerator (at the back of the glass shelf above the vegetable compartment).
- Always check the milk temperature before feeding.
- Repeated freeze/thaw cycles destroy the structure of the milk. Therefore, never refreeze breast milk. Never add freshly expressed milk to frozen milk.
- Plastic bottles and parts become brittle when frozen and may break when dropped.
- Do not use the breast milk from bottles or components that show signs of damage.

Storage guidelines for freshly expressed breast milk (for healthy term babies)			
Room temperature 16 to 25°C (60 to 77°F)	Refrigerator 4°C (39°F) or colder	Freezer -18°C (0°F) or colder	Breast milk thawed in the refrigerator
Up to 4 hours is best *Up to 6 hours for milk expressed under very clean conditions	Up to 3 days is best * Up to 5 days for milk expressed under very clean conditions	Up to 6 months is best * Up to 9 months for milk expressed under very clean conditions	At room temperature: Up to 2 hours Refrigerator: Up to 24 hours Do not refreeze!

I - TECHNICAL PARAMETERS

Model	RBRP-2024	Battery type	Li-Ion 1500mAh 3.7V
Input	5V DC, 1A		

Normal charging ambient temperature range of the product : 10-40°C

J - TROUBLESHOOTING

PROBLEMS	SOLUTIONS
Milk doesn't flow	- Check that the breastpump is able to suck and that all parts have been correctly assembled - Don't worry - try to relax. The amount of milk can vary from day to day; you may have to wait a while on your first few tries. - You could apply a soothing compress or put a warm, damp flannel on your breasts for a few minutes. This will help to release the milk and soothe your breasts. - Check nipple alignment, pause the pump and remove it from breast. Realign the pump and center nipple in the insert/cup tunnel - You could also swap to the other breast - If the valve is damaged, replace it with the provided spare one
Breast pain while pumping	- Make sure to press the pump firmly against the breast - Tighten your bra to hold pump firmly against the breast - Decrease suction strength - You may not use the correct insert size (refer to user manual to choose correct size of insert) - If pain persists, see your healthcare professional or a breastfeeding expert
Decreased (low) suction	- Increase suction setting by pressing + - Check that all connected parts are secure - Press the pump firmly against the breast to create a gentle seal - If the valve is damaged, replace it with the provided spare one
Stop pumping	- Pump stops automatically after 30 minutes. - Light press  will pause the pump, press again to resume
Pump is not charging	- Ensure the USB type-C cable is fully plugged into the pump charging port - Try a different charging source
Parts show signs of wear	Avoid putting the parts into contact with detergents that may damage the plastic.

If the above solutions still do not solve the problem, please contact Bebeconfort customer care

K - WARRANTY AND DISPOSAL

WARRANTY

Bebeconfort will replace or repair (at our option) your unit free of charge from 24 months from the date of purchase if the unit is defective in workmanship or materials, when used in normal conditions and in accordance with our user manual. Contact Consumer Service within 24 months of purchase from an authorised retailer.

WARNING

Opening or tampering with this device will void the manufacturer's warranty.

Not respecting charging recommendations (page 5) will void the warranty

DISPOSAL

Disposal of the device at the end of its operating life, separate the parts of the breastpump and dispose of according to the following instructions.

Breast shield, connector, and accessories The parts are made of plastics that are not harmful to the environment when disposed of as household waste. Recycle or dispose of according to local regulations.

Pump unit and USB cable Do not dispose of electric or electronic equipment together with unsorted municipal waste, but collect it separately.

Dispose of the waste equipment according to local regulations. Inquire at the point of sale or contact your local authority for appropriate collection points for waste equipment.

The separate collection and recycling of your waste equipment at the time of disposal will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

END OF LIFE LITHIUM BATTERY REMOVAL - HOW TO REMOVE BATTERY

You can protect the environment by removing and recycling the battery from this product before disposal.

Make sure the motor unit is disconnected from the USB charging cable before removing the battery.

CAUTION

Hazardous materials present in electric/electronic equipment may harm human health and the environment if the waste equipment is not disposed of properly.

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DOREL FRANCE S.A.S. Z.I. - 9 bd du Poitou, BP 905, 49309 Cholet Cedex, FRANCE